Reg. No. RU - 13020030036
ISO 9001:2015

FOCT P NCO 9001-2015

CE 0598

GB THE INSTRUCTION FOR USE FOR FILTERING HALF MASKS RK 9000 SERIES

Model	Protection level	Occupational Exposure Limit (OEL)	Comments
RK 9010	FFP1 R D	4	No valve
RK 9011	FFP1 R D	4	Valve
RK 9020	FFP2 R D	12	No valve
RK 9021	FFP2 R D	12	Valve
RK 9030	FFP3 R D	50	No valve
RK 9031	FFP3 R D	50	Valve

Description of placing the mask. Instructions for wearing

1. Remove the mask from the packaging. Check the condition of the mask for any holes and damage to the filter material. Pull the lower loop of the head strap (Fig. 1) and place it to the back of your head (Fig. 2).

2. Put on the filtering half mask. Hold the respirator with one hand and place the upper loop of the head strap to the top of your head with the other hand (Fig. 3). Adjust the elastic head straps and lock them with clamp. Form the noseclip around the nose (Fig. 4).

3. Check position and seal: Hold both hands around the mask, and exhale/inhale. If you feel any air escaping around the nose area, the mask should once again be correctly adjusted. If air leaks are noted around the line of contact, repeat and/or readjust the head straps, adjust mask position on the face to achieve a better seal. If no effective seal can be achieved, the respiratory protective device should not be used!

Storage

Storage or warehouse areas must have a normal climate. I.e. the air must be dry, cool and free from hazardous substances. Storage between -30°C and +70°C, atmospheric humidity <80%. Unpacked or damaged masks may no longer be used.

The storage period, under these conditions, is limited to 5 years.

Disposal

The filtering half mask should be disposed of following use in a safe and environmentally-friendly manner, in accordance with the national regulations.

IMPORTANT NOTES

Allocated protection factors may differ from country to country.

Statutory stipulations and regulations

2016/425:	Personal Protective Equipment (PPE) – Regulation (EU)
EN149:2001+A1:2009 EU type examination by:	Filtering half masks to protect against particles SGS Fimko Ltd. 0598
Production control according to Module D:	Notified body number: 0598 SGS Fimko Ltd. Takomotie 8 FI-00380 Helsinki
Official marks	

Manufacturer: Respiratori Kompleks Ltd.

3 Chekalova St., building 592, Leningradskaya obl., GP Im. Morozova, 188679, Russia.

+7 (812) 329-11-65; zakaz@rkszpe.ru; www.rkszpe.com

GB

Declaration of conformity to the PPE Regulation (EU) 2016/425 can be found at:
http://rkszpe.ru/respirator/rk9000/declaration/

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© «Respiratori Kompleks»